
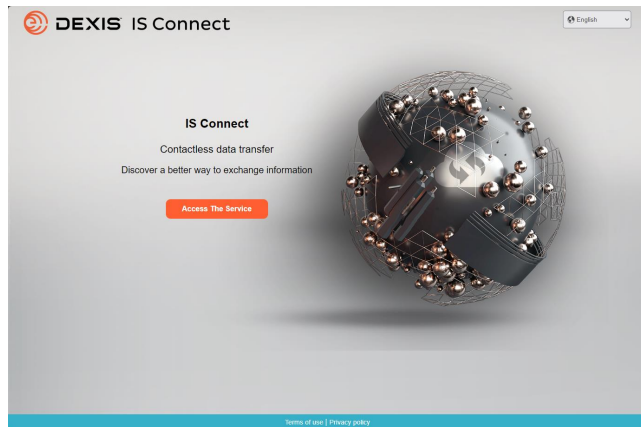
<u>English</u>	2
<u>French</u>	18
<u>Spanish</u>	34
<u>German</u>	50
<u>Portuguese</u>	66
<u>Italian</u>	82
<u>Swedish</u>	98
<u>Dutch</u>	114
<u>Japanese</u>	130

DEXIS-exocad integration setup


User accounts you need

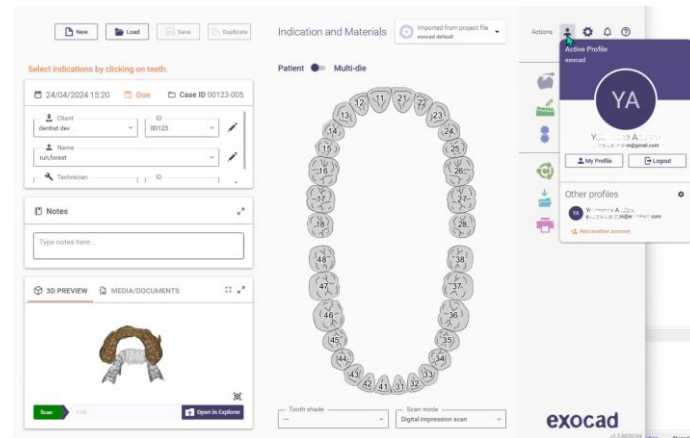☐ IS Connect account


☐ exocad DB account

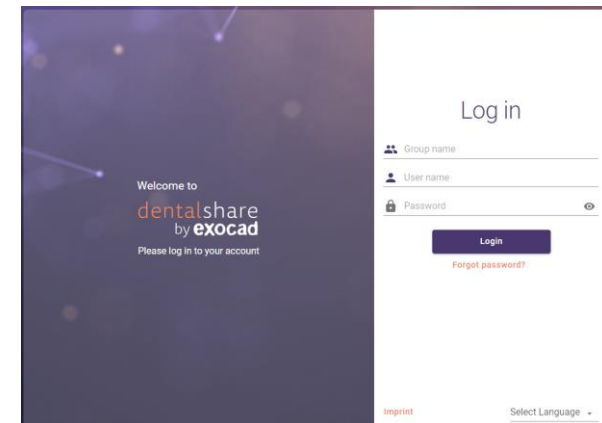☐ exocad dentalshare account

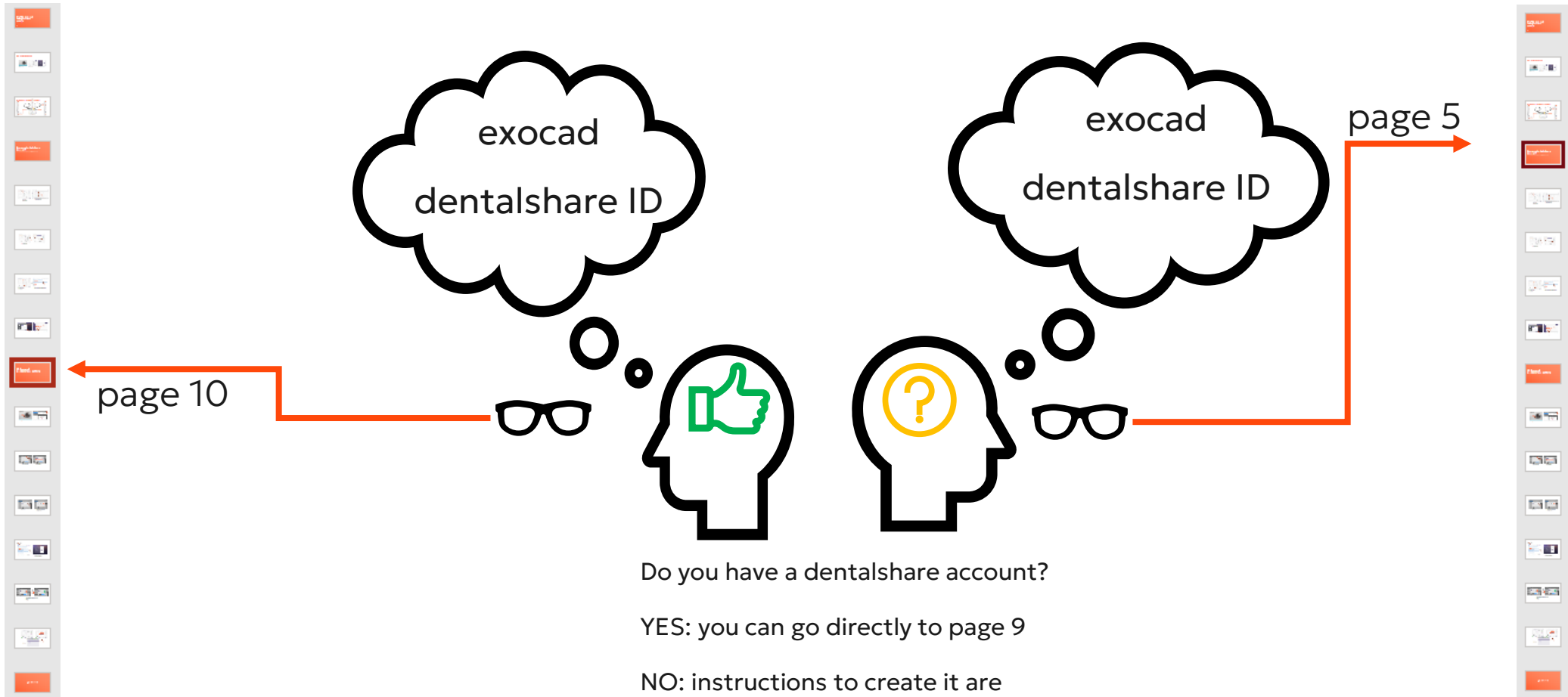Do you have a dentalshare account ?


Do you have a dentalshare account?

YES: you can go directly to page 9


NO: instructions to create it are provided page 4. You may also visit

https://wiki.exocad.com/wiki/index.php/Signing_up_to_Dentalshare

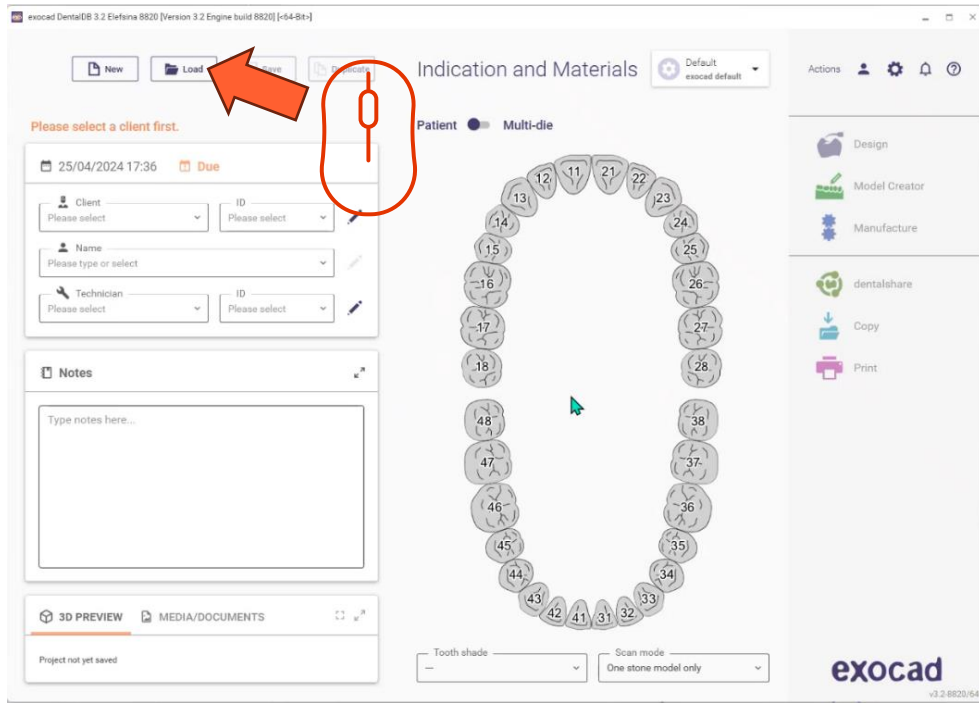
Create dentalshare account


- Open exocad DB
- Click on load button

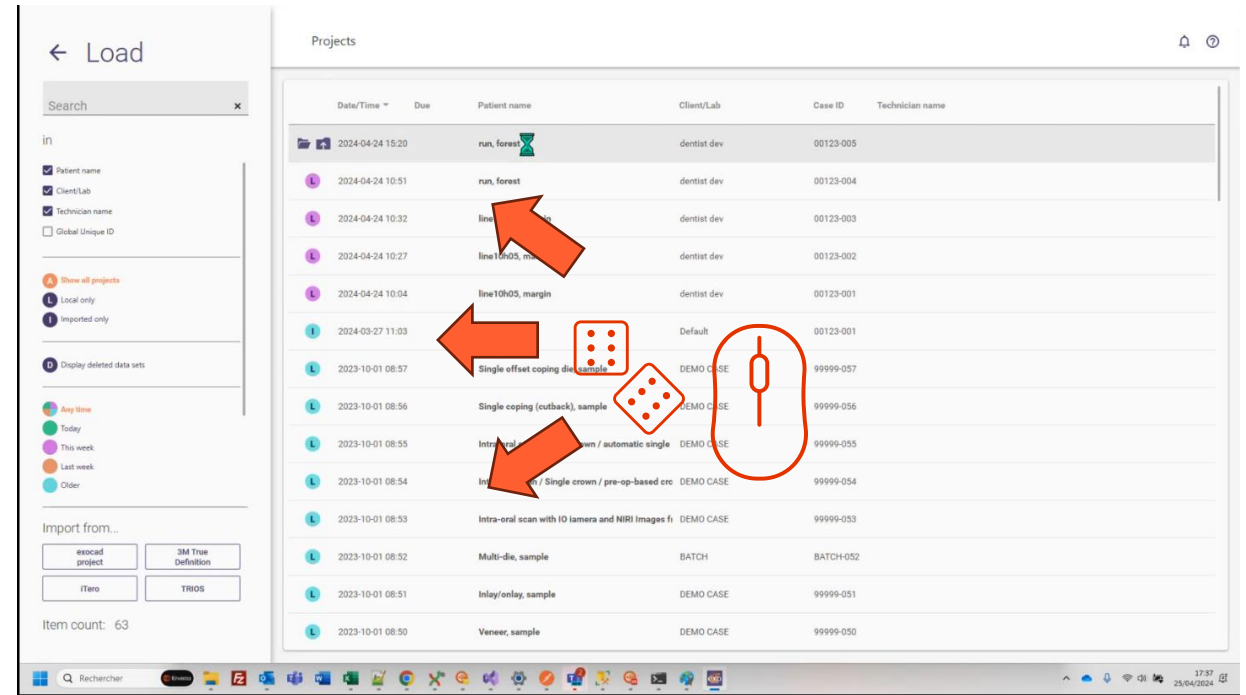
- Select any project and load it

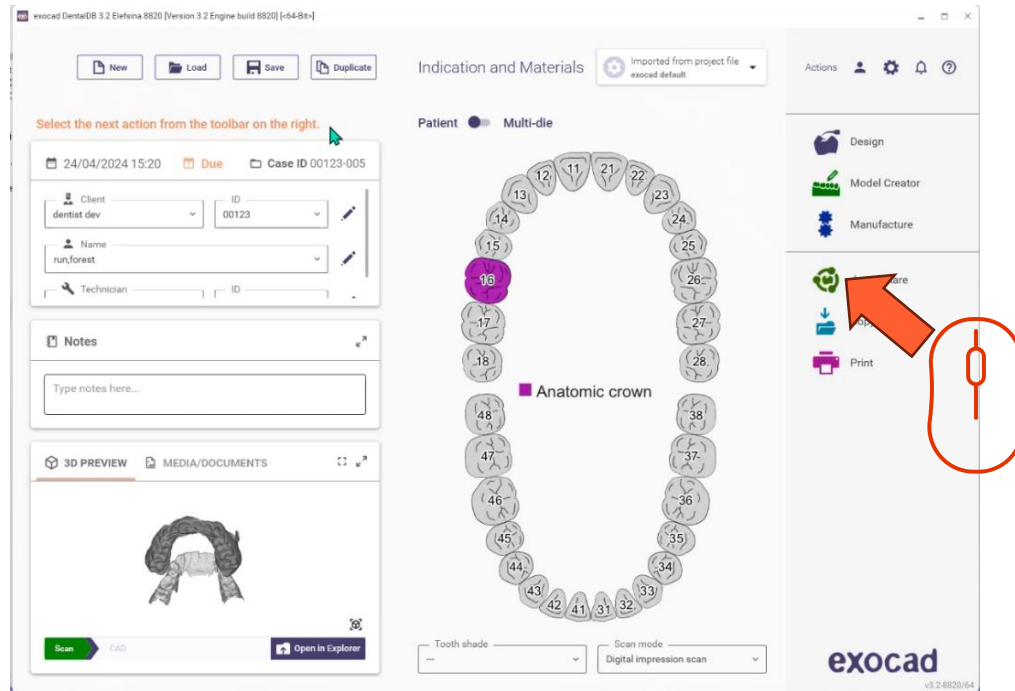- Click the dentalshare icon

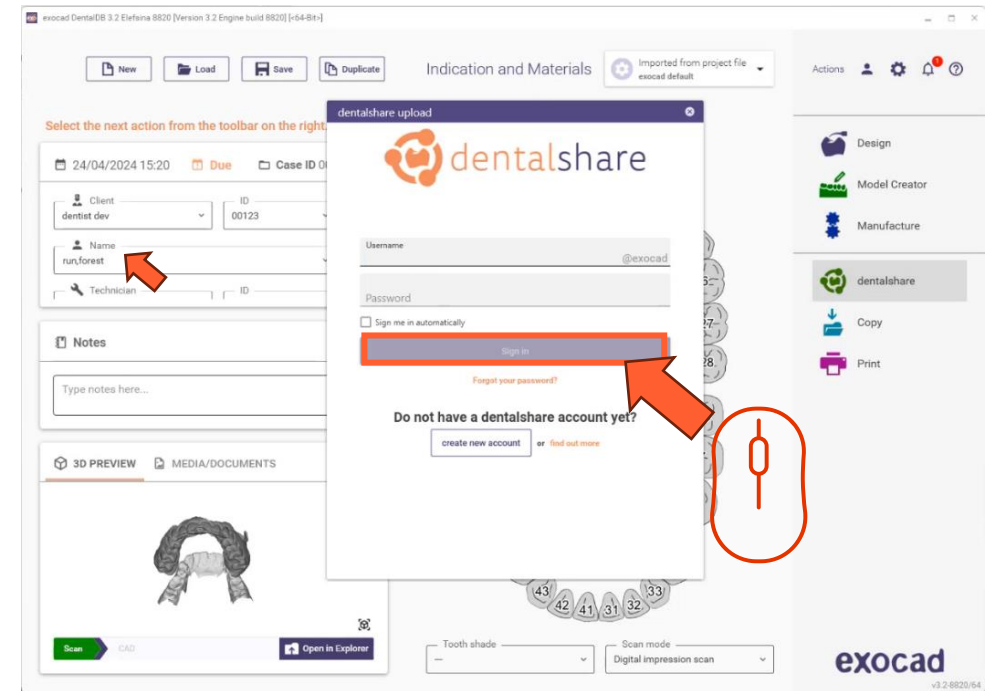
- Click on 'create account'

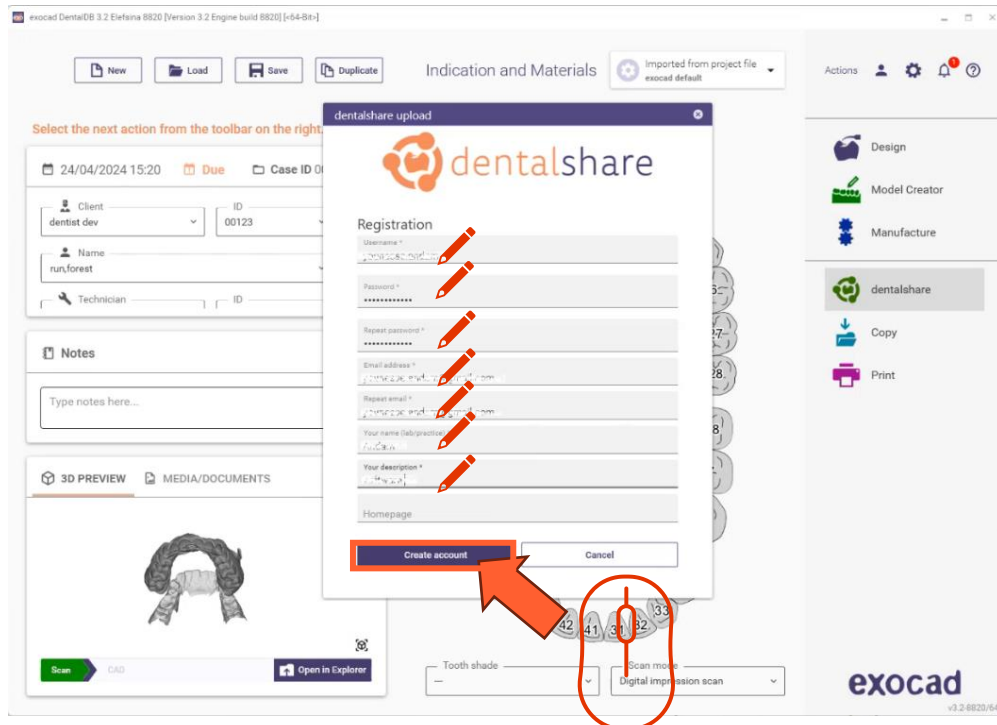
- Fill in requested information
- Click 'create account'
- An email is sent to your exocad account email

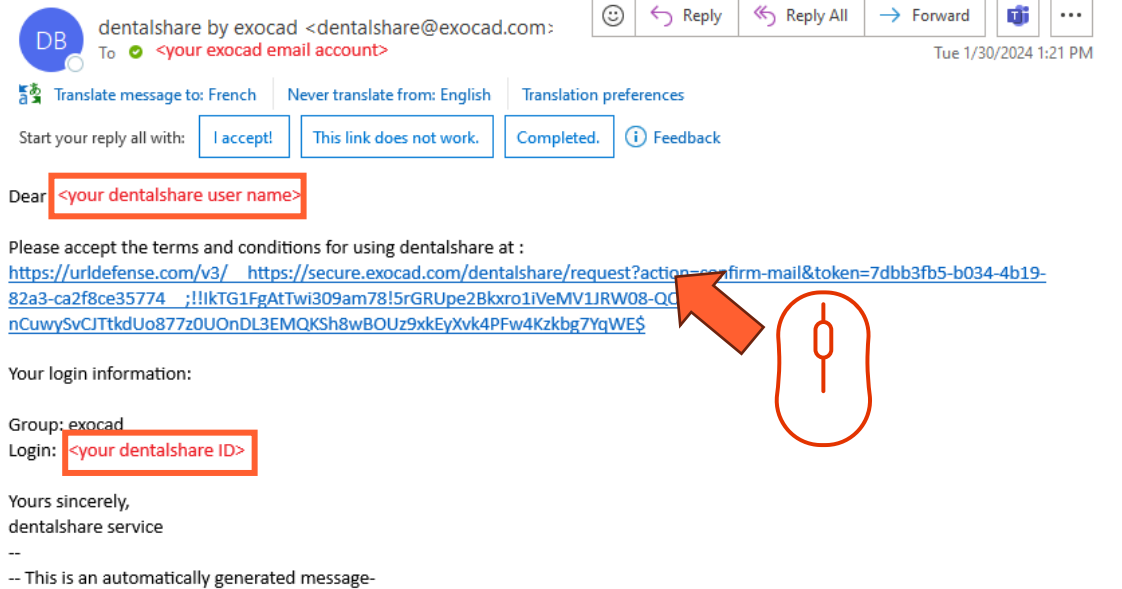
dentalshare: action required

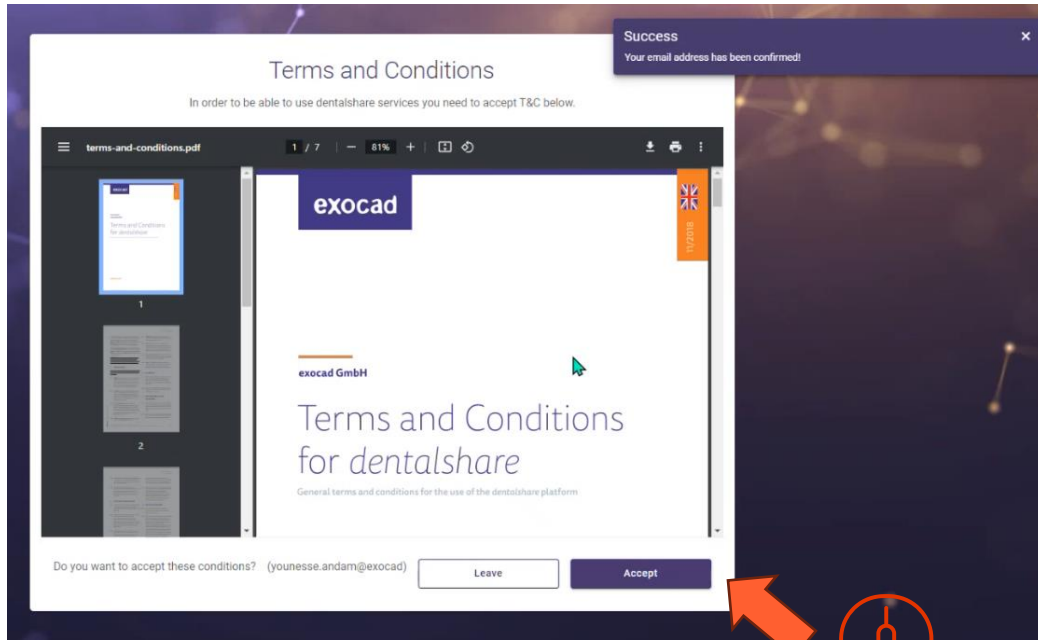- Click on link provided in the dentalshare email to accept dentalshare terms and conditions


- Accept terms and conditions

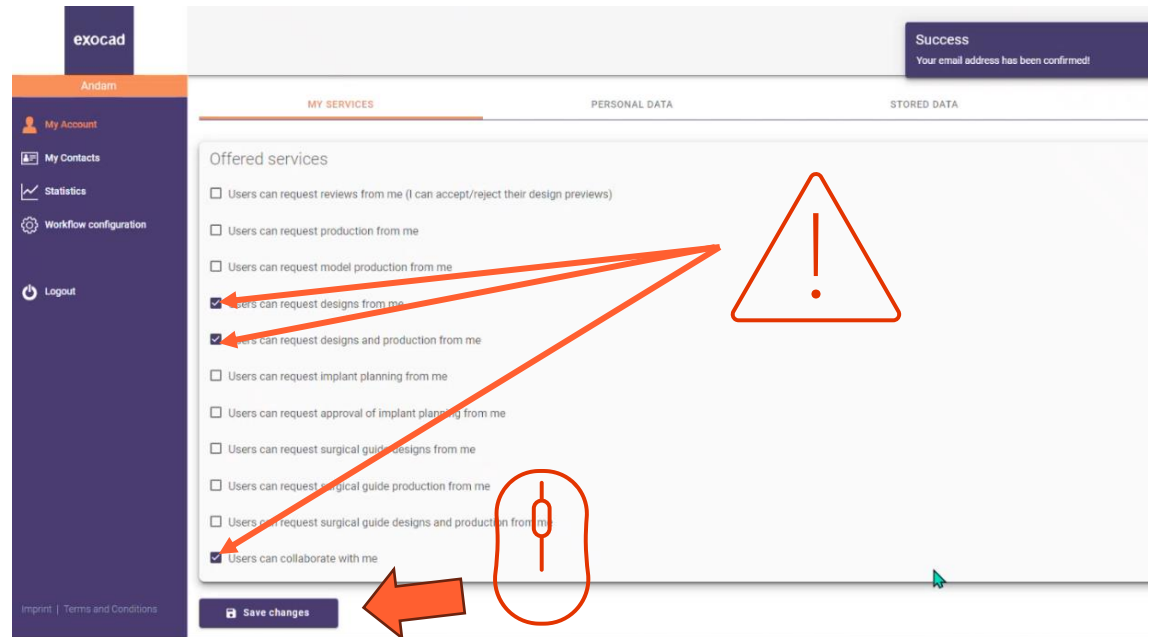
- Ensure your dentalshare account is configured to accept design requests
- Click on 'save changes'

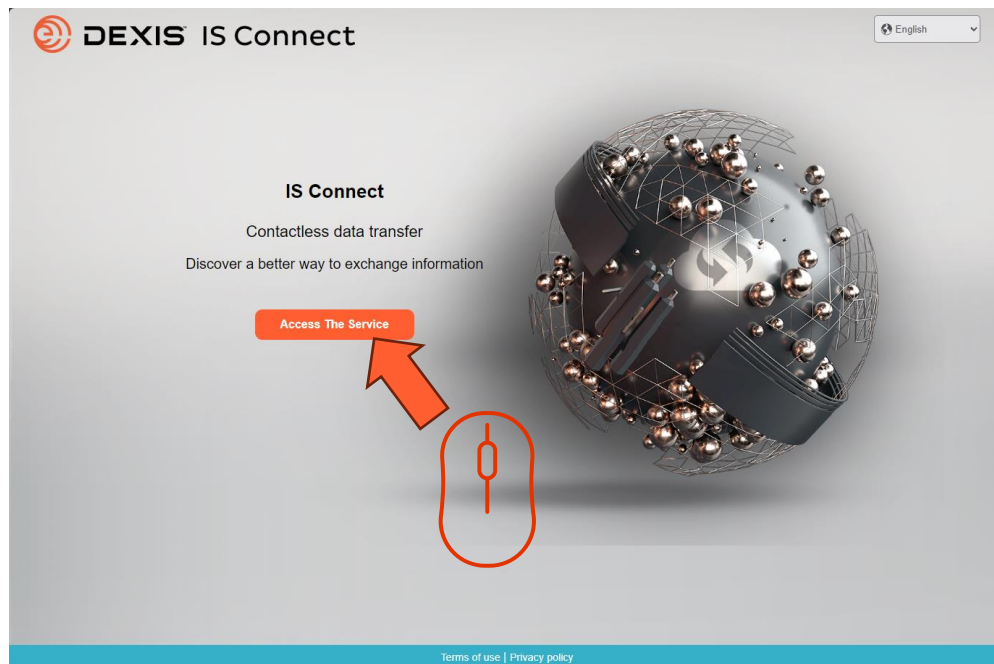
IS Connect dentalshare setting


- Sign in IS Connect

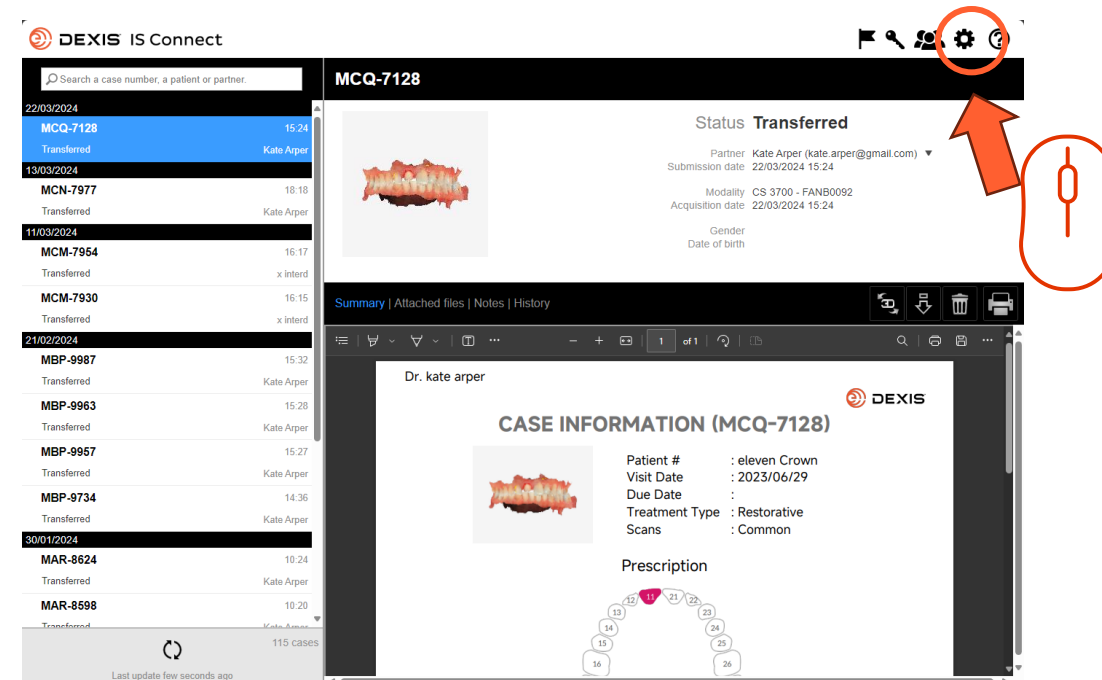
- Click on setting icon

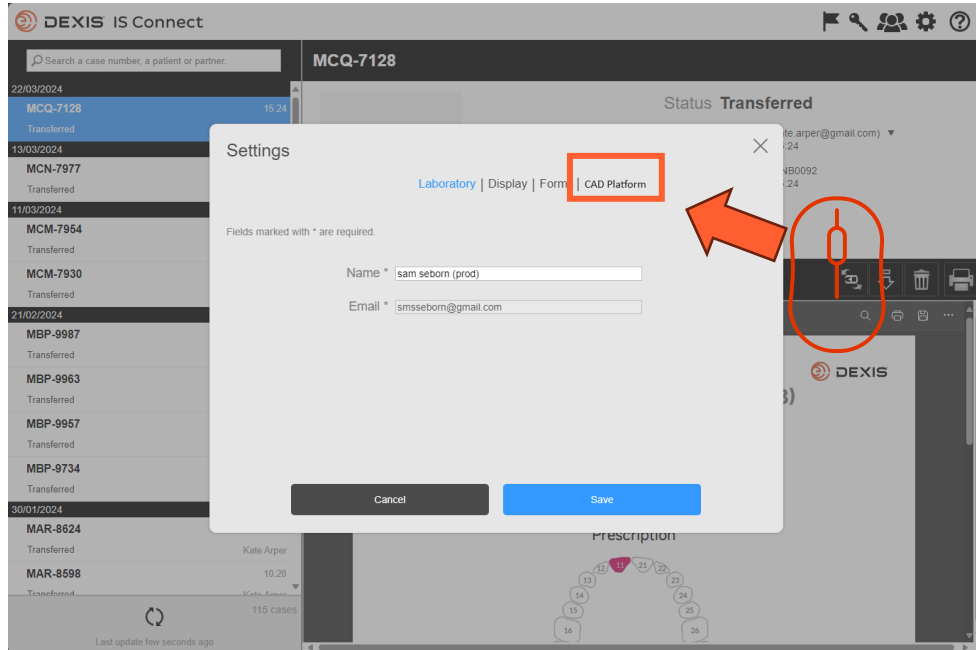
- Click on 'CAD Platform' tab

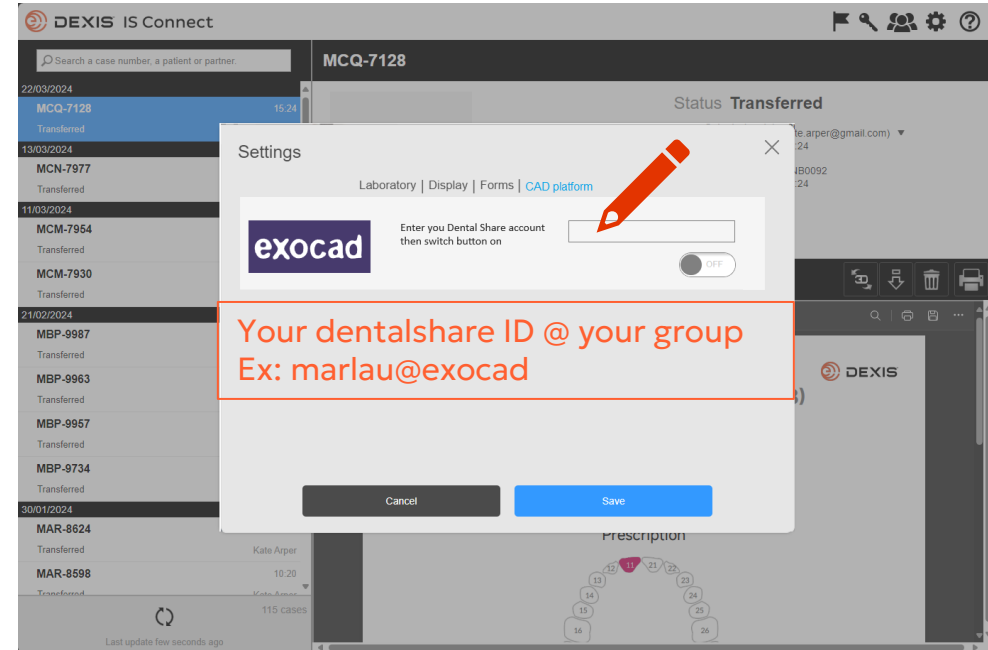- Type your dentalshare ID

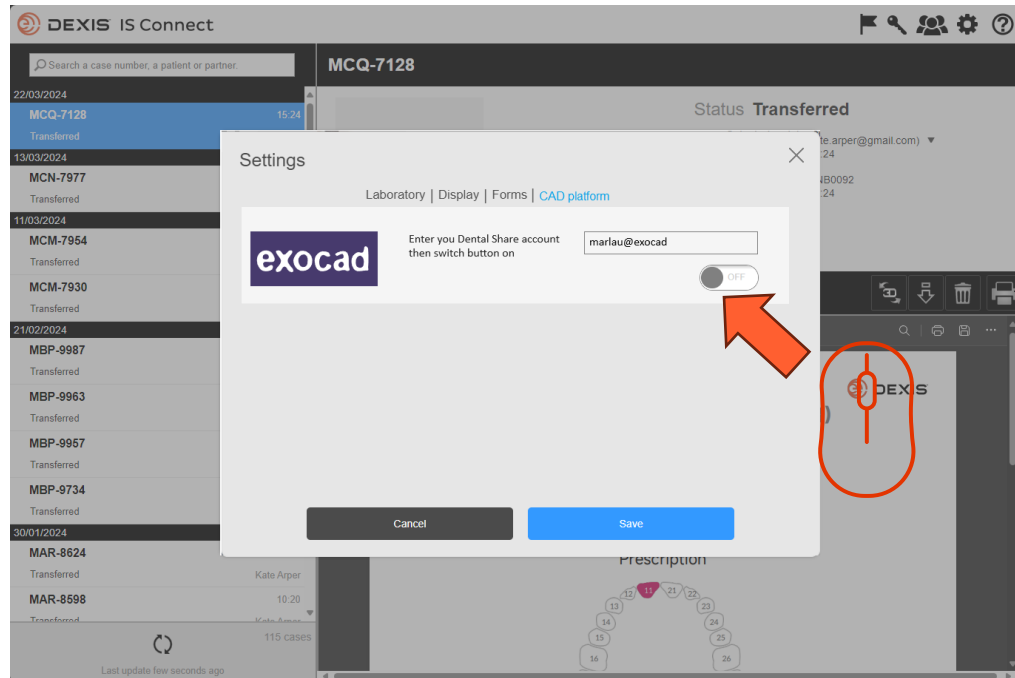
- Switch the button

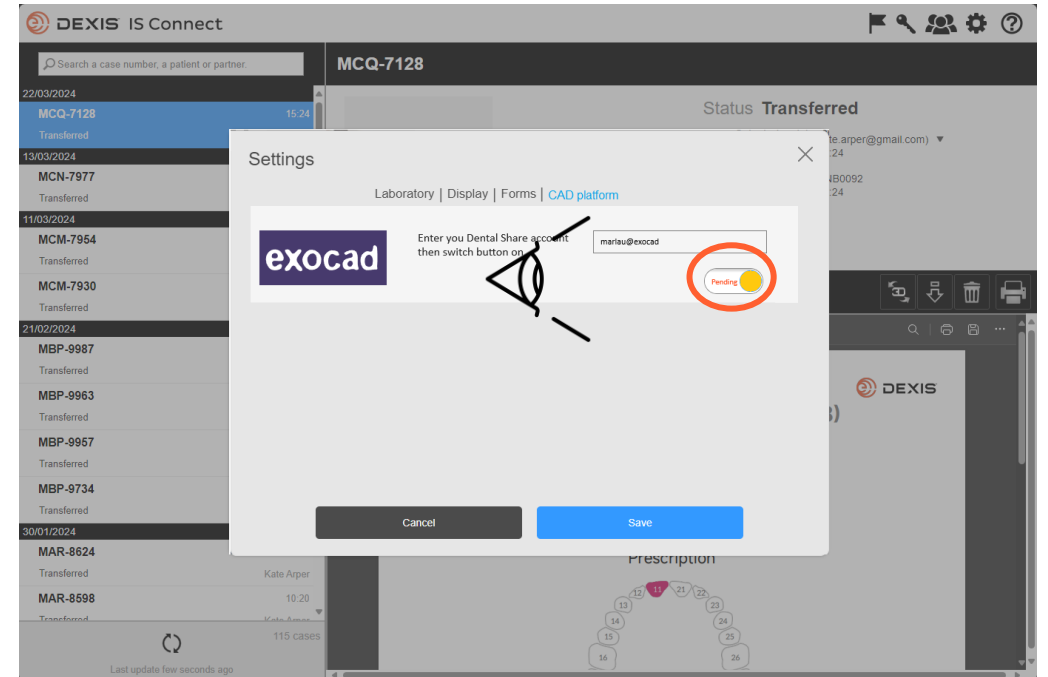- The button changes to 'pending' state
- You may quit IS Connect and wait for a dentalshare invitation

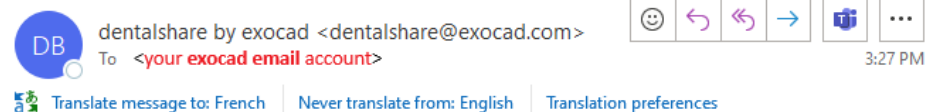dentalshare contact request

Dear <your dentalshare ID>

DEXIS IS Connect wants to offer a dentalshare community contact to you.

If you want to accept this contact, please click this link:

[https://urldefense.com/v3/https://secure.exocad.com/dentalshare/request?action=accept-contact&token=0f6bd346-dd7f-47c8-91f7-8fd032fd02ea;!!ikTG1FgAtTwi309am78!6urWMs_6iCZfyAwTxcVwGY_eugcCICBxJedErYEORJScjeid3kQUmKWPw4s8zWf4KWWmNoj_5gD05Sqibiv62VZzb3Ac\\$](https://urldefense.com/v3/https://secure.exocad.com/dentalshare/request?action=accept-contact&token=0f6bd346-dd7f-47c8-91f7-8fd032fd02ea;!!ikTG1FgAtTwi309am78!6urWMs_6iCZfyAwTxcVwGY_eugcCICBxJedErYEORJScjeid3kQUmKWPw4s8zWf4KWWmNoj_5gD05Sqibiv62VZzb3Ac$)

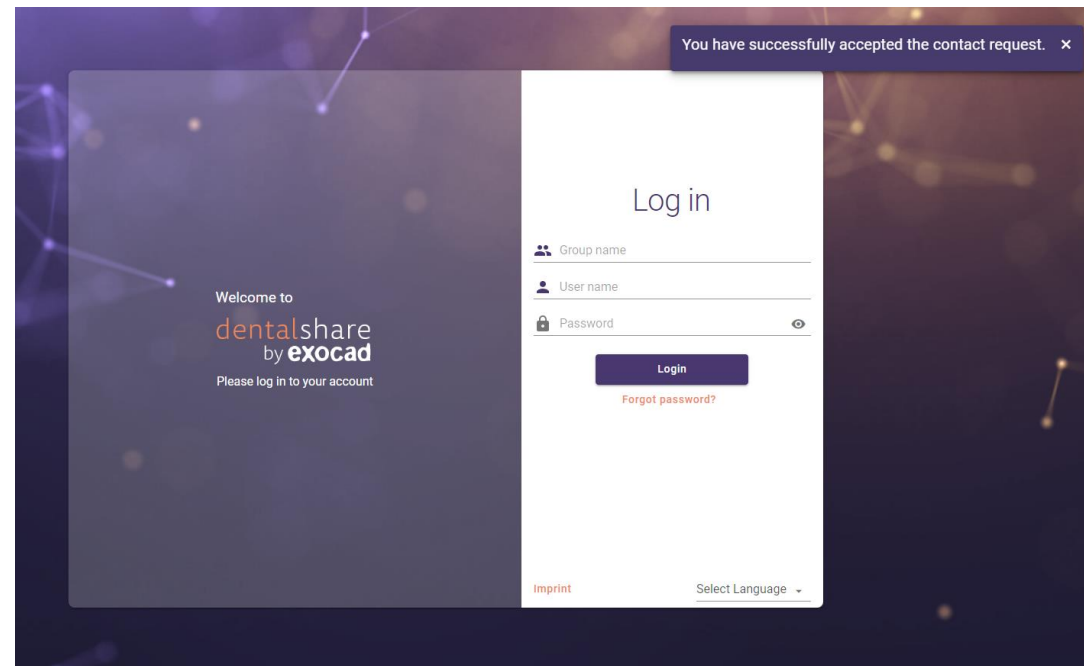
if you do not, please click this link:

[https://urldefense.com/v3/https://secure.exocad.com/dentalshare/request?action=decline-contact&token=0f6bd346-dd7f-47c8-91f7-8fd032fd02ea;!!ikTG1FgAtTwi309am78!6urWMs_6iCZfyAwTxcVwGY_eugcCICBxJedErYEORJScjeid3kQUmKWPw4s8zWf4KWWmNoj_5gD05Sqibiv62s4kiM30\\$](https://urldefense.com/v3/https://secure.exocad.com/dentalshare/request?action=decline-contact&token=0f6bd346-dd7f-47c8-91f7-8fd032fd02ea;!!ikTG1FgAtTwi309am78!6urWMs_6iCZfyAwTxcVwGY_eugcCICBxJedErYEORJScjeid3kQUmKWPw4s8zWf4KWWmNoj_5gD05Sqibiv62s4kiM30$)



Yours sincerely,
dentalshare service

-- This is an automatically generated message --


- You receive a dentalshare invitation from DEXIS IS Connect account
- Click on the link to accept it

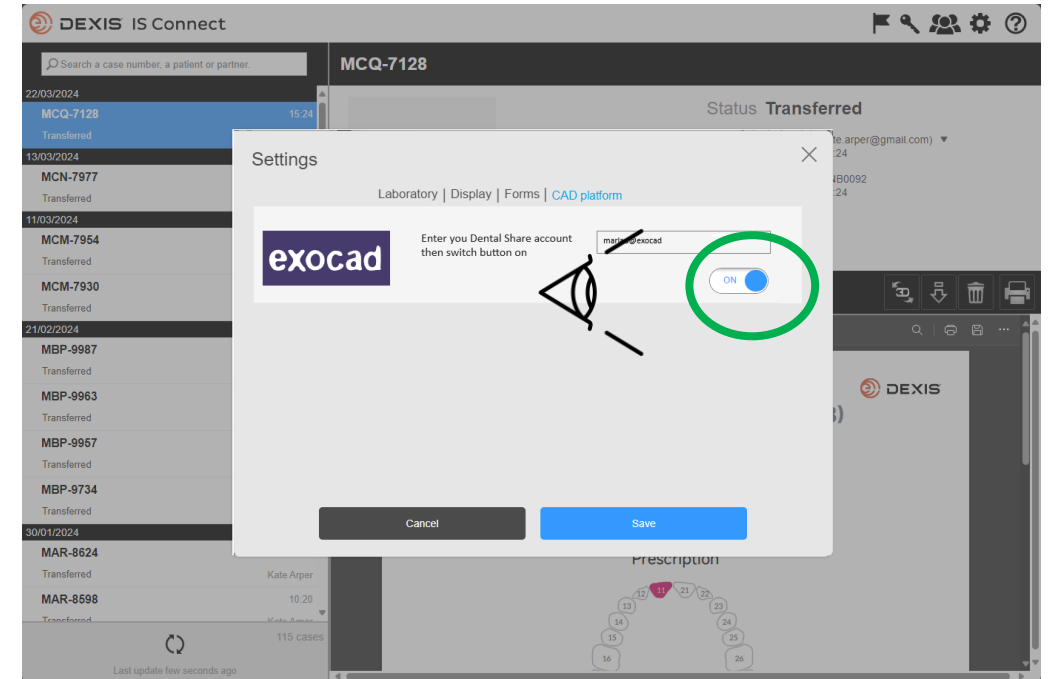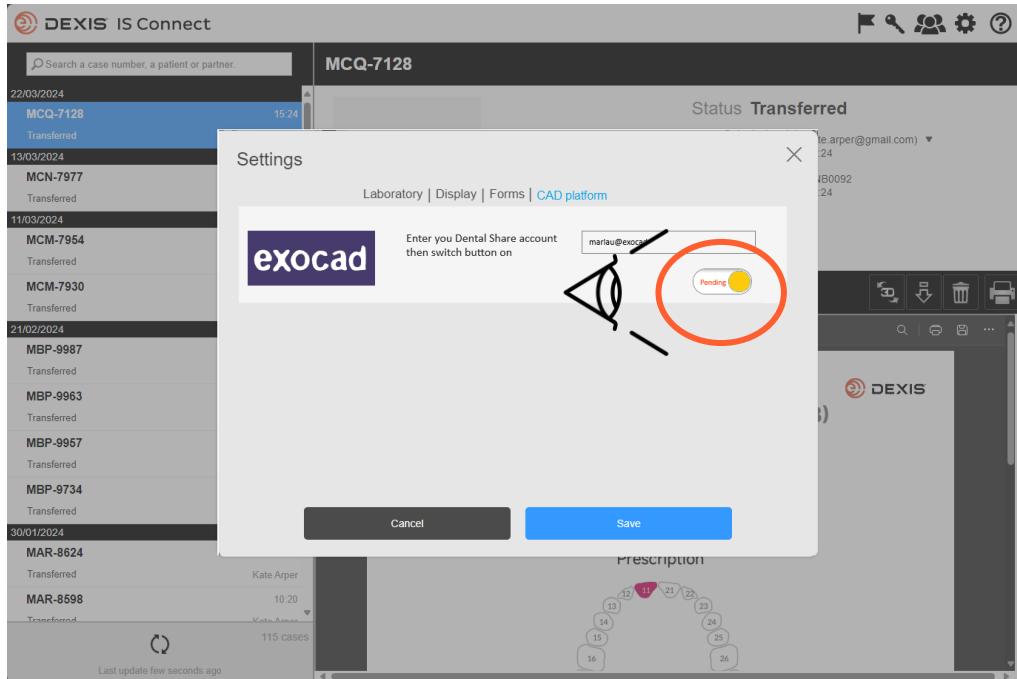
- You invitation is confirmed
- You may close the dentalshare screen

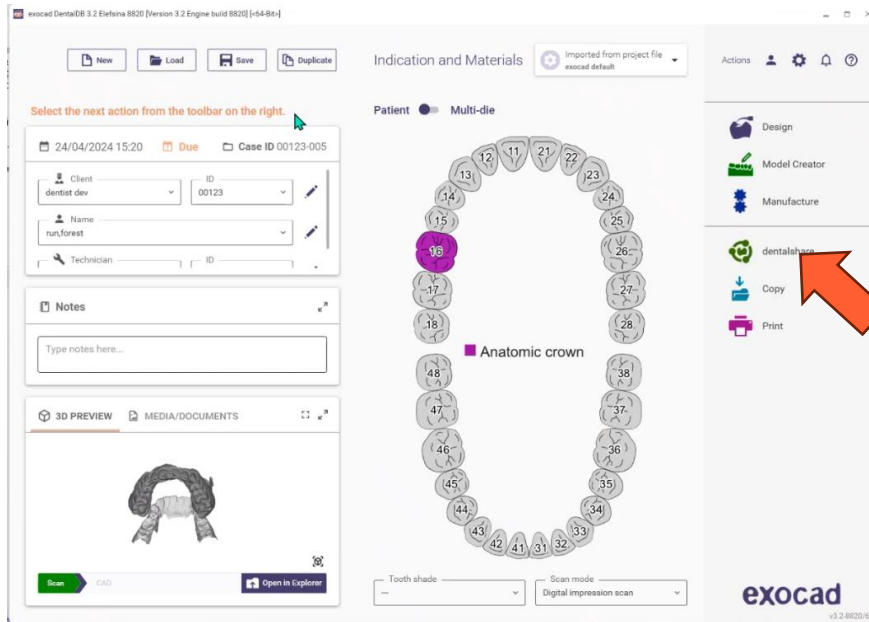- After you have accepted the dentalshare DEXIS IS Connect invitation, the exocad button in IS Connect setting will switch from pending to 'ON'
- A delay may be observed
- The integration is now registered

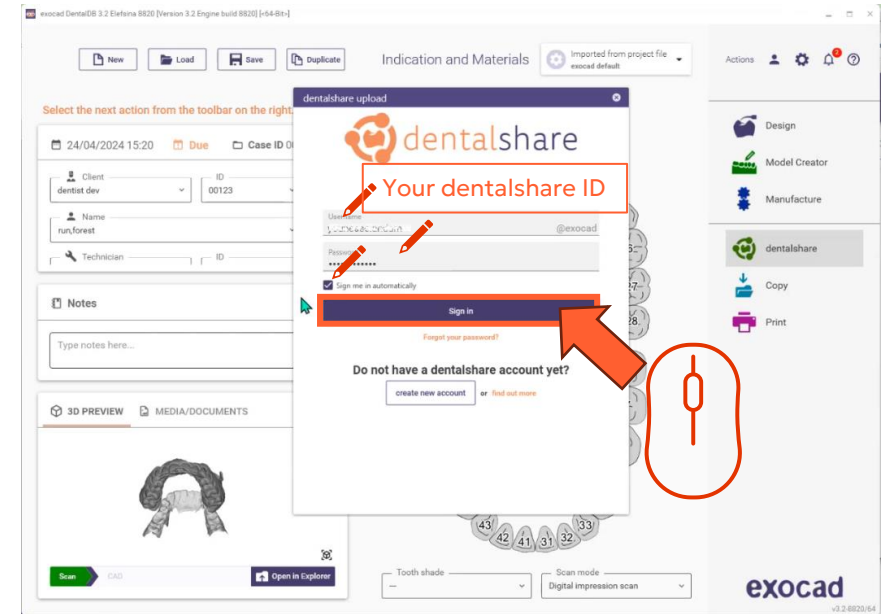
- You must be logged in dentalshare while in exocad DB to make the integration active

- Sign in dentalshare from exocad DB with your dentalshare ID and select 'sign in automatically'

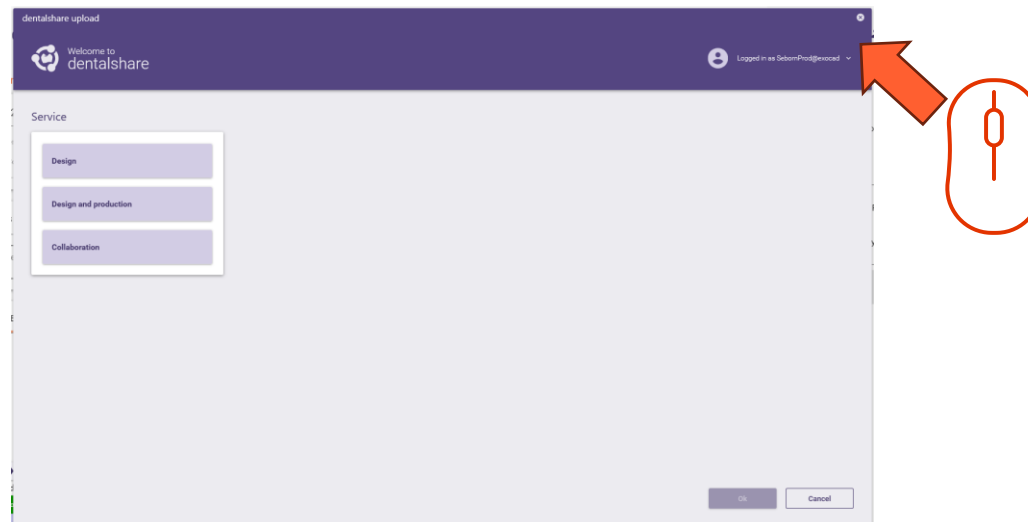- Dentalshare opens, you may close it
- IS Connect cases sent to your lab will now be delivered as dental project imported into your exocad DB

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